Description:

DiaQuant[®] pH 3.6–6.1 are non-bleeding test strips for use in dialysis. By using DiaQuant[®] pH 3.6–6.1, a residue check for hydroxyacetic acid subsequent to heat disinfection may be carried out safely and quickly.

Measuring range:

pH 3.6–6.1

Color gradation:

pH 3.6 · 4.1 · 4.4 · 4.7 · 5.0 · 5.3 · 5.6 · 6.1

Pack content:

100 test strips

Reaction principle:

The test strips contain a chemically bound pH indicator that changes from yellow to red in the pH range 3.6 to 6.1.

General information:

Only remove the required number of test strips from the container, and then immediately close the container securely afterwards. Do not touch the test field. Ideally, the color evaluation should be carried out by diffuse daylight. Artificial light can lead to difficulties or an incorrect color reading.

Instructions for use:

1. Insert the test strip into the sample solution and keep it there until the color of the test field stops changing. If using lightly buffered solutions, insert test strip for a longer period (1–15 min).
2. Shake off any excess liquid.
3. Compare the test field with the color scale. Take the value which matches closest with the colored test field (reading accuracy: ± 0.1 pH).

Quality control:

The correct functioning of the test strips may be verified by using commercially available pH buffer solutions.

Interferences:

No interferences are known.

Storage:


Avoid exposing the test strips to sunlight or moisture. Keep container cool and dry (storage temperature between 4 °C and 30 °C). If stored correctly, the test strips may be used until the use-by-date printed on the packaging.

Additional information:

The test strip container stopper contains a non-toxic drying agent. If swallowed, drink plenty of water.

Disposal:

Dispose of used test strips as domestic waste.

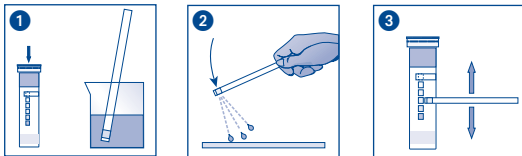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